










Immunology Control Low

REF:606 001 1x1 ml
REF:606 002 2x1 ml

SYMBOLS IN PRODUCT LABELLING	
	Authorised Representative
	For in-vitro diagnostic use
	Batch Code/Lot number
	Catalogue Number
	Consult instructions for use
	Use by/Expiration Date
	CAUTION. Consult instructions for use
	Manufactured by
	(Xi) - Irritant

Intended Use

Accuracy control for the determination of proteins in serum by turbidimetry and nephelometry.

composition

A dilution of human plasma with phosphate buffered saline.
Contains stabilizers and 0.095 g% sodium azide .

Storage and Stability

The expiry date of the product at +2 to +8 °C is listed on the label.
After first opening the control can be used for 6 weeks if stored tightly closed at +2 to +8 °C after use.
Do not freeze.

Precautions and Warnings

- 1.For "In Vitro Diagnostic Use"
- 2.Each individual donation intended for use in manufacturing of protein control serum was tested for hepatitis B surface antigen (HBsAg), anti-hepatitis C virus (anti-HCV) and anti-HIV1 and HIV2 by FDA required test. Only donations with negative findings were used for its manufacture. Nevertheless every product obtained from human body fluids should be handled with appropriate care in accordance with recommended procedures for biohazardous materials since absence of infectious agents can never be proven .
- 3.Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact skin or mucous membranes !

Assigned Values

The values are lot dependent. Actual values are printed on the Assay Value Sheet .



Egyptian Company for Biotechnology (S.A.E)

Obour city industrial area. block 20008 piece 19 A. Cairo. Egypt.

Tel: +202 4489 2248 - Fax: +202 4489 2247

www.spectrum-diagnostics.com

E-mail: info@spectrum-diagnostics.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



IFUFTI39

Rev.(2), 1/8/2022

